

Recommended Power Settings with Electrodes

Procedures	Pure Cut	Blend Cut	Coagulation	Electrode
Spay - Cat		20-35	30	ES60/ES60R
Dental Extraction - Cat			25	ES60/ES60R
Onychectomy - Cat Declawing		12-20	30	ES60/ES60R
Onychectomy - Cat Declawing older cat		15-25	30	ES60/ES60R
Enucleation - Removal of a Cats eye		13	20	ES60/ES60R
Neuter - Cat		20-25	20	ES60/ES60R ES01/ES01R
Gingervectomy small dog (lump removal gum)		17	30	ES26-8
Gingervectomy medium dog (lump removal gum)		20	30	ES26-8
Small wart Dessication Dog			25	ES01
Toenail Cautery Med to Large Dog			35	ES20R
Soft pallet (soft tissue was trimmed)		15	20	ES02 or ES60
Stenotic Nares (nose job)		15	20	ES02 or ES60
Neuter (small dog 8 lbs)	20-25		15	ES02 or ES60
Neuter (medium dog 30 lbs)	20-30		20	ES02 or ES60