

# High-Temperature Cauteries From A Name You Trust

*Aaron Medical manufactures more types of cauteries than anyone else in the world! Therefore, Aaron has the ability to put the right cautery in the physician's hand regardless of the procedure.*

**Fine Tip** (AA01 -AA11 -AA25) used for pinpoint hemostasis and for evacuation of a subungual hematoma.

**Loop Tip** (AA03-AA27) a popular choice where controlling more diffuse bleeders is required.

**Vasectomy Tip** (AA21 -AA29) specifically designed for precise hemostasis of the vas deferens while performing a vasectomy.

**Two-Inch (5.1 cm) Flexible Shaft – Loop Tip** (AA05), **Fine Tip** (AA17), excellent for sculpting woven grafts during bypass surgery or when working in recessed areas.

**Five-Inch (12.7 cm) Flexible Shaft Loop Tip** (AA09) deeply recessed areas, (AA09L) includes a lamp for additional light at the recessed operative site.

The patented (US Patent #5,688,265) Aaron RAM™ high-temperature cauteries combine faster from zero to full temperature plus a four-year shelf life. These cauteries, manufactured in the USA, are quite simply the best ever made! All cauteries are individually packaged sterile, ten per shelf pack.



7100 30th Avenue North • St. Petersburg, FL 33710-2902  
U.S. Phone 800-537-2790 • Fax 800-323-1640  
International Phone (727) 384-2323 • Fax (727) 347-9144  
[www.aaronmed.com](http://www.aaronmed.com) • E-mail = [customer.service@aaronmed.com](mailto:customer.service@aaronmed.com)

# Aaron Medical's Line of High-Temperature Cauteries

## CATALOG NUMBER

## DESCRIPTION

- AA01 Fine Tip
- AA25 Fine Tip Micro
- AA11 Adjustable Fine Tip

Fine tip high-temperature cauteries are routinely used in cardiovascular, thoracic, oculoplastic, orthopaedic and hand-surgery procedures where precise hemostasis is required. Also excellent for evacuation of subungual hematoma. Micro versions are manufactured with AAA alkaline batteries for lighter weight and smaller diameter tubes.

- AA03 Loop Tip
- AA27 Loop Tip Micro


Loop tip high-temperature cauteries are popular choices for general surgery, ENT and orthopaedic surgical procedures where controlling more diffuse bleeders is required. Micro versions are manufactured with AAA alkaline batteries for lighter weight and smaller diameter tubes.

- AA21 Vasectomy Tip
- AA29 Vasectomy Tip Micro

Vasectomy tip high-temperature cauteries are designed for precise hemostasis of the vas deferens. This sutureless vasectomy is a simplified technique performed in the office or operating room. The elongated fine tip can also be used for sculpting woven grafts. Micro versions are manufactured with AAA alkaline batteries for lighter weight and smaller diameter tubes.

- AA17 2" Fine Tip
- AA05 2" Loop Tip
- AA09 5" Loop Tip
- AA09L 5" Lighted Loop Tip

Two-inch loop tip, fine tip, and five-inch loop tip cauteries are used in oral and nasal surgery, pacemaker implantation, laminectomies and any other surgical procedure requiring extended reach. Useful for pacemaker implants and colostomy opening procedures, removing pedunculated warts and other skin lesions.

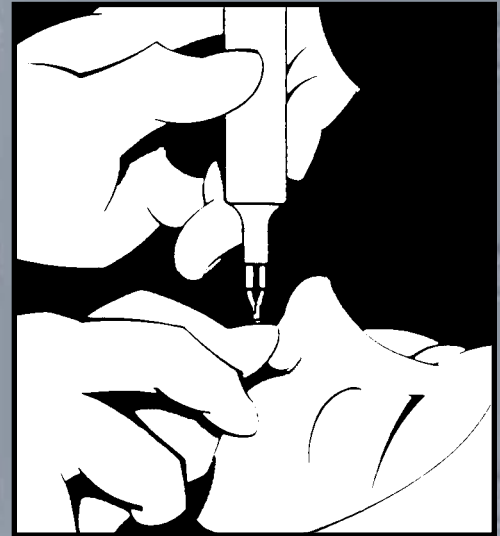

## CONVERSION CHART

*Cordless Surgical Cauteries*

Xomed/Treace Concept	AARON
84-40000	AA02
4050	AA00
84-41000	AA00 or AA04
84-41250	AA92
84-41500	AA21
84-42000	AA01
84-43000	AA03
84-44000	AA05
84-45000	AA17
4500	AA09
84-46000	AA11
84-48000	AA45
84-48010	AA35
NA	AA25
NA	AA27
NA	AA29

Distributed By:

- Sterile
- Maneuverable
- Disposable
- Cost Effective
- Cordless
- Choice of Tip Styles
- Versatile
- Convenient "Pressure Sensitive" Switch
- Lightweight