

Nerve Locator-Stimulator From A Name You Trust



7100 30th Avenue North
St. Petersburg, FL 33710-2902
800-537-2790 • Fax 800-323-1640
(727) 384-2323 • Fax (727) 347-9144
www.aaronmed.com
E-mail = customer.service@aaronmed.com

Nerve Locator-Stimulator Description

CATALOG NUMBER**DESCRIPTION**

- 0003Y Neuro-Pulse™
Nerve Locator - Stimulator
With Yellow LED
- 0003 Neuro-Pulse™
Nerve Locator - Stimulator
With Red LED

Nerve Locator - Stimulators

The Neuro-Pulse™ Surgical Nerve Locator was designed for identifying exposed motor nerves while at the same time reducing the possibility of accidental nerve damage or severance.


To assure the surgeon that a completed circuit has been achieved and the instrument is operative, the LED Light (conveniently built into the end) illuminates each time the probe needle comes in contact with exposed tissue. This light can also be used to test the battery by touching the grounding needle to the probe needle.

For various nerve size requirements, the Neuro-Pulse™ has three output settings – 1/2mA at 3 Volts DC, 1mA at 3 Volts DC, and 2mA at 3 Volts DC.

The Neuro-Pulse™ lightweight, sterile construction allows the surgeon maneuverability with an unobstructed view.

The self-powered Neuro-Pulse™ is portable and requires no external power source. Batteries can be tested by simply touching the rear of the ground needle with the probe needle.

The Neuro-Pulse™ is ideal for Orthopaedic and Neurosurgical procedures as well as head, neck, hand, plastic, or facial applications.

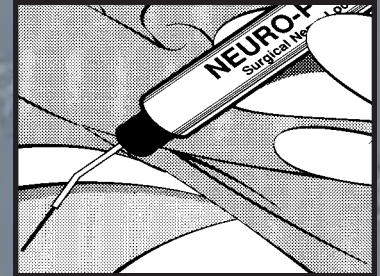

Distributed By: