

Portable Cryosurgical System



...Patient friendly, convenient to use and provides efficient and safe patient outcomes

- Results comparable to liquid nitrogen¹
- Convenient and easy to use
- Effective cure rate
- Minimal treatment time for patients , 40 seconds or less
- Designed with the lowest (GWP) Global Warming Impact
- Reusable can with disposable applicators
- Estimated 50+ treatments per box system
- Low cost treatments with profitable reimbursement rates
- No Hazmat Fee

Treat your patient in 40 seconds or less

SKIN LESION TYPE	RECOMMENDED FREEZING TIMES
Warts: plantaris & vulgaris	40 seconds
Warts: plana	20 seconds
Molluscum Contagiosum	20 seconds
Skin tags	40 seconds
Actinic keratosis (facial)	15 seconds
Actinic keratosis (non-facial)	40 seconds
Ano-genital warts (condyloma acuminatum)	40 seconds
Lentigines (facial)	15 seconds
Lentigines (non-facial)	40 seconds
Seborrheic keratosis	40 seconds



1. Caballero Martinez F, et al. Cutaneous cryosurgery in family medicine: dimethyl-ether-propane spray vs liquid nitrogen. Aten Primaria (1996); 18(5): 211-216

Choose Your Configuration

ITEM #	PRODUCT DESCRIPTION
CRY2	IceOut Portable Cryosurgical System, (2) 80 ml aerosol dispensers and (60) 2 mm applicators
CRY5	IceOut Portable Cryosurgical System, (2) 80 ml aerosol dispensers 2 cans and (50) 5 mm applicators
CRY25	IceOut Portable Cryosurgical System, (2) 80 ml aerosol dispensers and (60) 2mm & 5 mm applicators

Expand practice offerings with IceOut

Patient Convenience

Refer Less Patients Out

Increase Reimbursement

Easy as one, two, three...

1. Insert applicator and charge by pressing down until fluid starts to form.

2. Hold for 15 seconds.

3. Apply frozen applicator on effected area for the indication time desired.



Reimbursements

INDICATION	CPT CODE	CPT CODE DESCRIPTION	CMS PHYSICIAN FEE SCHEDULE CY 2017	ESTIMATED LEVEL OF REIMBURSEMENT/KIT	
				(36 Application)	(70 Application)
Acrochordon (Skin Tags)	11200	Removal of skin tags <w/15	\$90.08	\$3,242.88	\$6,305.60
Acrochordon (Skin Tags)	11201	Remove skin tags add-on	\$19.38	\$697.68	\$1,356.60
Actinic Keratosis	17000	Destruct premalignant lesion	\$67.83	\$2,441.88	\$4,748.10
Actinic Keratosis	17003	Destruct premalignant lesion 2-14	\$5.74	\$206.64	\$401.80
Common Warts	17110	Destruct b9 lesion 1-14	\$112.69	\$4,056.84	\$7,888.30
Flat Warts	17110	Destruct b9 lesion 1-14	\$112.69	\$4,056.84	\$7,888.30
Lentigo	17110	Destruct b9 lesion 1-14	\$112.69	\$4,056.84	\$7,888.30
Molluscum Contagiosum	17110	Destruct b9 lesion 1-14	\$112.69	\$4,056.84	\$7,888.30
Plantar Warts	17110	Destruct b9 lesion 1-14	\$112.69	\$4,056.84	\$7,888.30
Seborrheic Keratosis	17110	Destruct b9 lesion 1-14	\$112.69	\$4,056.84	\$7,888.30
Seborrheic Keratosis	17111	Destruct lesion 15 or more	\$133.86	\$4,818.96	\$9,370.20
Genital Warts (Male)	54050	Destruction penis lesion(s), simple	\$136.02	\$4,896.72	\$9,521.40
Genital Warts (Male)	54056	Cryosurgery penis lesion(s)	\$145.35	\$5,232.60	\$10,174.50
Genital Warts (Male)	54065	Destruction of lesion(s) penis, extensive	\$224.30	\$8,074.80	\$15,701.00
Genital Warts (Female)	56501	Destroy vulva lesion(s), simple	\$133.51	\$4,806.36	\$9,345.70
Genital Warts (Female)	56515	Destroy vulva lesion(s), extensive	\$231.48	\$8,333.28	\$16,203.60

The CPT codes listed are representative of the principal codes used for cryotherapeutic treatments. This information is general coding information only. It is not reimbursement or legal advice nor is it advice about how to code or submit any particular claim for payment. It is the provider's responsibility to determine and submit appropriate codes. Before filing any claims, providers should verify current requirements and policies with individual payors. Actual payments will vary based on geographical location, payor type, facility/non-facility, and other factors. This coding and reimbursement information is subject to change without notice.

Source: CY2017 CMS Physician Fee Schedule, Searchable Medicare Physician Fee Schedule (MPFS); Data Last Modified: 04/10/2017